

# EFR KIT Easy Fixture Removal KIT (OSFRK)




Top panel components

Fixture Wrench  
FRDFE



For **TSII / III** **SSII / III** **USII / III** **Ultra-wide**

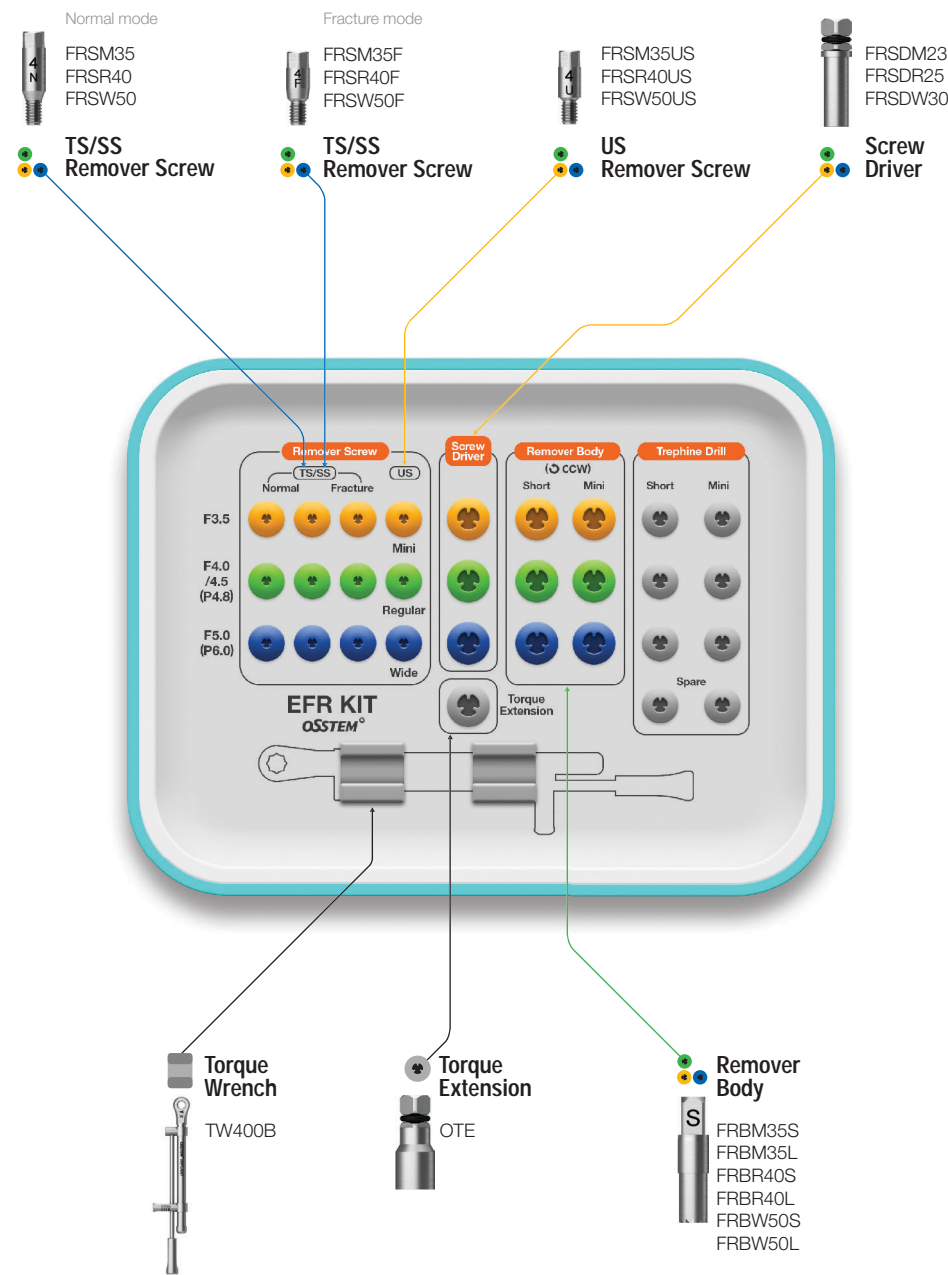

# EFR Full KIT Easy Fixture Removal Full KIT (OSFRFK)

• KIT has the same components as EFR KIT and can be put on competitors' components

Lower panel components

Fixture Wrench  
FRDFE



Torque Wrench  
TW400B



For **Nobel Biocare Active/Replace** / **Straumann Bone Level** / **Astra Osseo Speed TX**  
**3i Full OSSEOTITE Tapered Certain** / **Zimmer Tapered** / **Biohorizons Internal**





# EFR Full KIT Surgical Instruments

## Screw Driver

- Connects and fastens the remover screw to the fixture
- Recommended tightening torque : regular/wide 100Ncm, mini 80Ncm
- F = Fixture



F	Mini	Regular	Wide
	FRSDM23	FRSDR25	FRSDW30

## Torque Wrench

- Connect with screw driver to fasten and remover body to remove the fixture
- Applies up to 400Ncm of torque (markers at 80/100/200/300/400Ncm)
- Torque by pulling the bar back until reaching the desired torque value
- Clean and sterilize for storage



TW400B

## Remover Body

- Connects to a failed fixture via the remover screw and by applying counterclock wise torque, removes the implant
- Available in different sizes to match the diameters of the fixture to be removed
- ※ Disposable; do not re-use
- F = Fixture



F	Mini	Regular	Only for osstem Wide	Only for overseas companies Wide	Ultra-wide
Short	FRBM35S	FRBR40S	FRBW50S	FRBW57S	FRBUW60S
Long	FRBM35L	FRBR40L	FRBW50L	FRBW57L	FRBUW60L

## Fixture Wrench

- Removes implants from the remover body after removing the fixture from the bone



FRDFE

## Torque Extension

- Extends the length of the screw driver and remover body (by 10mm)



OTE