Prosthetic KIT

User Manual

OSSTEM[®]
IMPLANT

OSSTEM[®]
IMPLANT

2460

Storage Condition :
Dry place at room temperature (1°C~30°C)
Caution, Consult
accompanying
Documents

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Manual for the Prosthetic KIT

■ Product Description

The Prosthetic KIT consists of the surgical instruments required for performing a prosthetic operation following the grafting of an Osstem Implant (US, SS, TS System). For detailed directions, refer to the prosthetic catalogue provided by OSSTEM IMPLANT Co., Ltd.

■ Cleaning the KIT

Clean the treatment tools after use as follows :

1. After treatment, clean the tools using the conventional method, i.e., by soaking them in alcohol.
2. After cleaning the tools with distilled water or under running water, remove any remaining blood stains and foreign matter completely.
3. Remove moisture completely using a dry cloth or a heat fan.
4. Insert and secure the moisture-free treatment tools into the correct slots of the KIT.
5. After setting the tools, sterilize (132°C for 15 minutes) the KIT in an Autoclave and store at room temperature.

■ Caution During Use and Cleaning

1. The recommended usage frequency for the Driver is 50 times. The recommended Torque is equal to or less than 30Ncm.
2. The recommended usage frequency for the Torque Wrench is 50 times.
3. Detach all used instruments after operation and clean, dry, and keep them under normal temperature. For the cleaning method for the torque wrench, refer to the manual provided at the lower end of the KIT.
4. Do not leave the instruments in a place vulnerable to contamination.
5. Be sure to sterilize these medical tools in the Autoclave before use. (132°C, 15 minutes)
6. Do not use hydrogen peroxide as disinfectant and cleanser (may cause damage or discoloration of the TiN coating, Laser Marking and color coding).
7. This product comes with a ONE (1) year warranty from the date of its purchase.

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OBELIS S.A.