# Monitor Osseointegration. Enhance decisions about when to load.




- Reduces Treatment Time
- Manages Challenging Cases



# **Guidance and Predictability**

- Raise the implant success ratio with accurate diagnostic.
- Confidence to decide the optimal loading time.

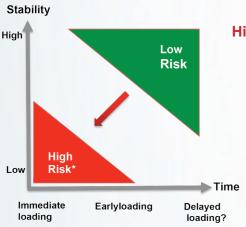


### **High Risk Area\* Sucess Ratio:**

- Bruxism 71.7%
- Diabetes 71.4%
- Immediate Loading 77.5%
- Iliac Crest Block + GBR + Sinus Graft 50%
- Surgeons <5 years experience had 5x failure</li>



# **Optimized Patient Care**


- Determines optimal loading protocol.
- The higher the ISQ Value, the higher the stability of the implant.

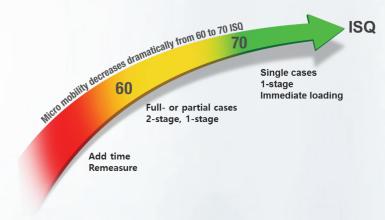

ISQ Standard Calibrated™


# **High Accuracy Measurement Results**

### **Minimum Variance**

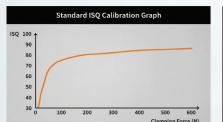

\*ISC International Standard Calibration

# **Optimal Physical Fit**

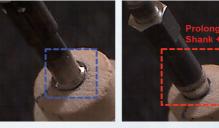

IS3 MulTipeg™ (#74) vs.Osstell Smartpeg (#50)

# Reusable MulTipeg™

- Up to 20 times without any deviation.
- Reusable, auto-clavable, calibrated, and compatible with all major implant systems.

# MulTipeg™ (Titanium)

- Reusable
- Auto-clavable
- Calibrated



# MulTipeg™ Driver

- Reusable
- Autoclavable
- Stainless Steel



# **Ease of Use**

- · Hand held instrument with single key operation.
- Clear numeric displays on both sides and long-lasting rechargeable batteries.



# **153** ™ System



# **Product System and Practical Use**

The IS3™ system consists of a hand-held instrument and a reusable MulTipeg™. A charger is delivered together with the instrument as well as a stainless steel driver for attaching the MulTipeg™ to the implant.



## STEP 1

Select the correct  $\underline{\text{MulTipeg}}^{\text{TM}}$  for the implant being measured.

### STEP 2

Attach the MulTipeg™ Using the driver (approximately 6-8 Ncm of torgue.





### STEP 3

Turn on the instrument and hold the tip of the instrument close to the top of the  $MulTipeg^{TM}$ .

### STEP 4

The instrument emits a beeping sound when it starts measuring and another sound when the ISQ value is presented on the display a second later.



# MulTipeg™ Assortment

Bredent

BTI

**BTK** 

C

Camlog

Conelog

C-Tech

ripeg Assortinein			
Α	D	J	Р
ADIN	Dentack	J Dental Care	Paltop
AIRES	Dentegris		Phibo
Alpha-Bio	Dentium	K	Prodent Italia
Anker	DEVA	Keystone	
Ankylos	DIO	Klockner	R
Anthogyr		Konmet	Resista
Argon	E		
Astra Tech	Easy Implant	L	S
Avinent	Edierre	Lasak	Schutz Dental
		Leader	SIC
В	G		Southern Implants
BEGO	Global D	M	Spiral Tech
BioHorizons	GMI	Medentika	Straumann
Biomet/3i		Megagen	Sweden & Martina
Bionnovation	Н	MIS	
Biotech Dental	Hahn		Т
BluSkyBio	Hiossen Implant	N	Tag Dental
Bredent		Neo Biotech	Thommen

Implance
Implant Direct
Implant Swiss
Implantvision
Intra-lock

Neodent
Neoss
Nobel Biocare
NucleOss

0

X

Ζ

Xive

OCO Biomedical Zimmer
Osstem Zuga

TRI Dental implants