


Mount & Screw

Simple Mount

- Select appropriate mount according to the fixture platform
- Use a 1.2 hex driver (fastened manually)
- ※ Disposable; re-use is not allowed
- P = Platform


- M Mini
- R Regular
- W Wide

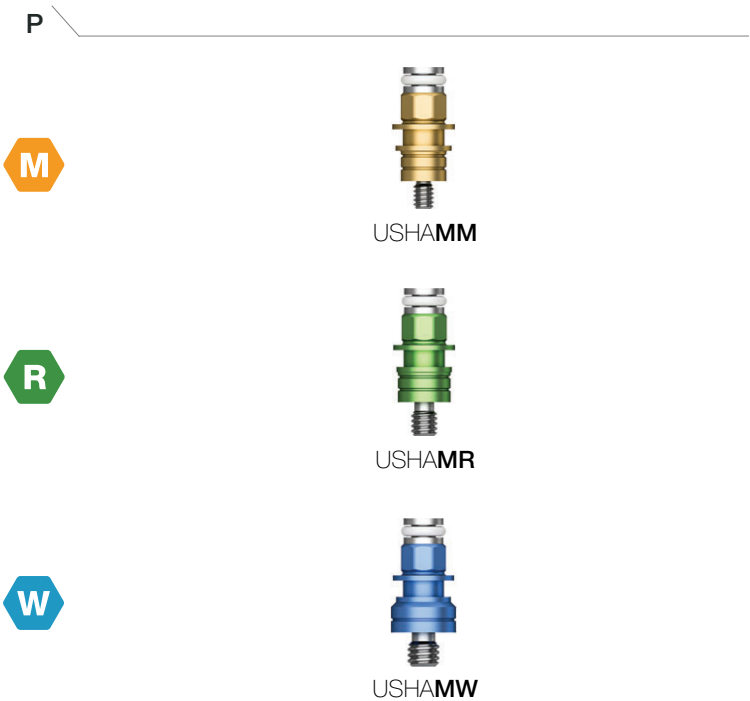

Cover Screw

- Select appropriate mount according to the fixture platform
- Hex driver: 0.9 (for mini components only) or 1.2 (torque manually)
- P = Platform


- M Mini
- R Regular
- W Wide

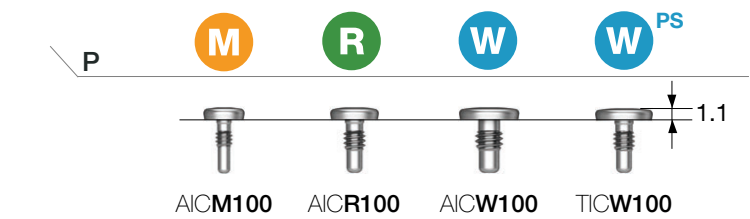

Headless Cover Screw

- Use when there is insufficient soft tissue
- Hex driver: 0.9 (for mini components only) or 1.2 (torque manually)
- P = Platform


- M Mini
- R Regular
- W Wide

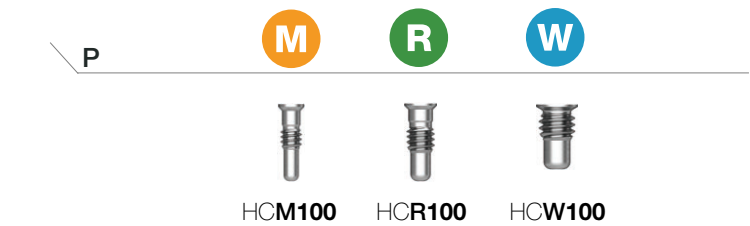

Healing Abutment

- Use a 1.2 hex driver
- Recommended tightening torque: 5 to 8 Ncm

- M Mini
- R Regular
- W Wide

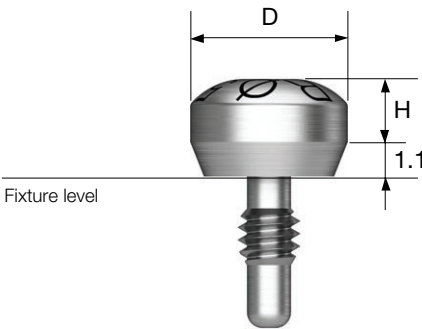

D \ H	2.0	3.0	4.0	5.5	7.0
Ø 4.0	-	AIHM403	-	AIHM405	-
Ø 5.0	-	AIHM503	-	AIHM505	-
D \ H	2.0	3.0	4.0	5.5	7.0
Ø 4.1 One Piece	-	AIOHR403	-	AIOHR405	AIOHR407
Ø 4.1 Two Piece	-	AIHR403	-	AIHR405	AIHR407
Ø 5.0	AIHR502	AIHR503	AIHR504	AIHR505	AIHR507
Ø 6.0	AIHR602	AIHR603	AIHR604	AIHR605	AIHR607
D \ H	2.0	3.0	4.0	5.5	7.0
Ø 5.1 One Piece	-	AIOHW503	-	AIOHW505	-
Ø 5.1 Two Piece	-	AIHW503	-	AIHW505	-
Ø 6.0	AIHW602	AIHW603	AIHW604	AIHW605	-
Ø 7.0	AIHW702	AIHW703	AIHW704	AIHW705	-
Ø 6.0 PS	-	TIHW603	-	TIHW605	-